STERIL BEAUTY

Ref. AS004ZSM

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INDEX

- 1. AUTOCLAVE USE AND APPLICATIONS**
- 2. SAFETY**
 - 2.1 SAFETY MARKING
 - 2.2 SAFETY DEVICES
 - 2.3 SAFETY NOTICES
 - 2.4 DISPOSAL
- 3. TECHNICAL DATA**
- 4. OUTPUTS AND INDICATORS LIST**
- 5. UNPACKAGING**
- 6. ACCESSORIES**
- 7. INSTALLATION**
- 8. USE INSTRUCTION**
 - 8.1 WATER TANK LOAD
 - 8.2 LOADING OF THE MATERIALS INTO THE AUTOCLAVE
 - 8.3 STERILIZATION CYCLE
 - 8.4 WATER TANK DISCHARGING
- 9. STERILIZATION CYCLES**
 - 9.1 CYCLES BACKUPS ON USB KEY
- 10. MAINTENANCE**
- 11. ALARM MESSAGES**
- 12. SOLUTION TO OPERATING PROBLEMS**
 - 12.1 THE AUTOCLAVE DOES NOT DRY CORRECTLY
 - 12.2 THE AUTOCLAVE CHAMBER HAS TURNED WHITE
 - 12.3 THE AUTOCLAVE CHAMBER HAS GREEN STAINS
 - 12.4 THE STERILIZATION CYCLE INTERRUPTS
 - 12.5 STAINS ON INSTRUMENTS
- 13. PROCEDURE FOR SERVICE AND ASSISTANCE**

1

AUTOClave USE AND APPLICATIONS

The autoclave is suitable to sterilize instruments exclusively for esthetician or beauty salons.

SOLID INSTRUMENTS

Instruments with no cavities and no obstacles to the penetration of steam

max kg. 0,5
for each tray

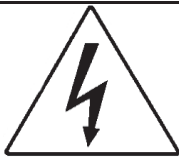
The autoclave shall only have to be used for the sterilization of tools and materials being compatible with the steam sterilization system. Always make sure that the loads that need to undergo sterilization can stand the temperatures scheduled for the selected cycle.

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WARNING
DISCONNECT THE
MAINS SUPPLY BEFORE
REMOVING THIS COVER

DISCONNECT
THE POWER
BEFORE
REMOVING
THE LID

EARTH
CONNECTION

2.2

Safety devices

The following safety devices are installed:

- Safety valve set at 2.4 bar +/-10%
- Resistance over temperature thermostats

2.3

Safety notes


- The manufacturer is liable for the marketed product in accordance with current regulations. The **manufacturer's liability** will expire when operations are carried out on the device, or a part of it, by unskilled personnel or using non-original spare parts.
- There should be no potential risk of explosion and/or fire in the room where the autoclave is installed.
- The autoclave should be installed in a special well-ventilated room.

2.4 Disposal

Refer to annex
Cod. 0Z00H0004

3
TECHNICAL DATA

MECHANICAL DATA	Working temperature	+5°C ÷ +40°C
	Maximum altitude	2.000 m a.m.s.l
	Max relative humidity at 30°C	80%
	Max relative humidity at 40°C	50%
	Dimensions of space occupied (L x H x P)(mm)	384 x 330 x 530
	Space occupied with open door	300 mm
	Weight (empty tanks)	27 kg.
	Weight (full tanks + room full)	31 kg.
	Potential sound level	< 50 db A
ELECTRICAL DATA	Power voltage	230 V a.c. +/-10 % single phase
	MAX power	1,5 kW
	Frequency	50 / 60 Hz
	Power cord	2x1.5mm ² + GROUND
	Fuses	8A F
CHAMBER	MAX working pressure	2.4 bar (relativi)
	MAX temperature	138 °C
	Material	Inox AISI 304
	Size (mm)	Ø 164 X 320 mm
WATER TANK	Volume	2.2 l
	Useful tank volume	MAX 1.2÷1.3 l
	Usable cycles	7
	Material	polietilene

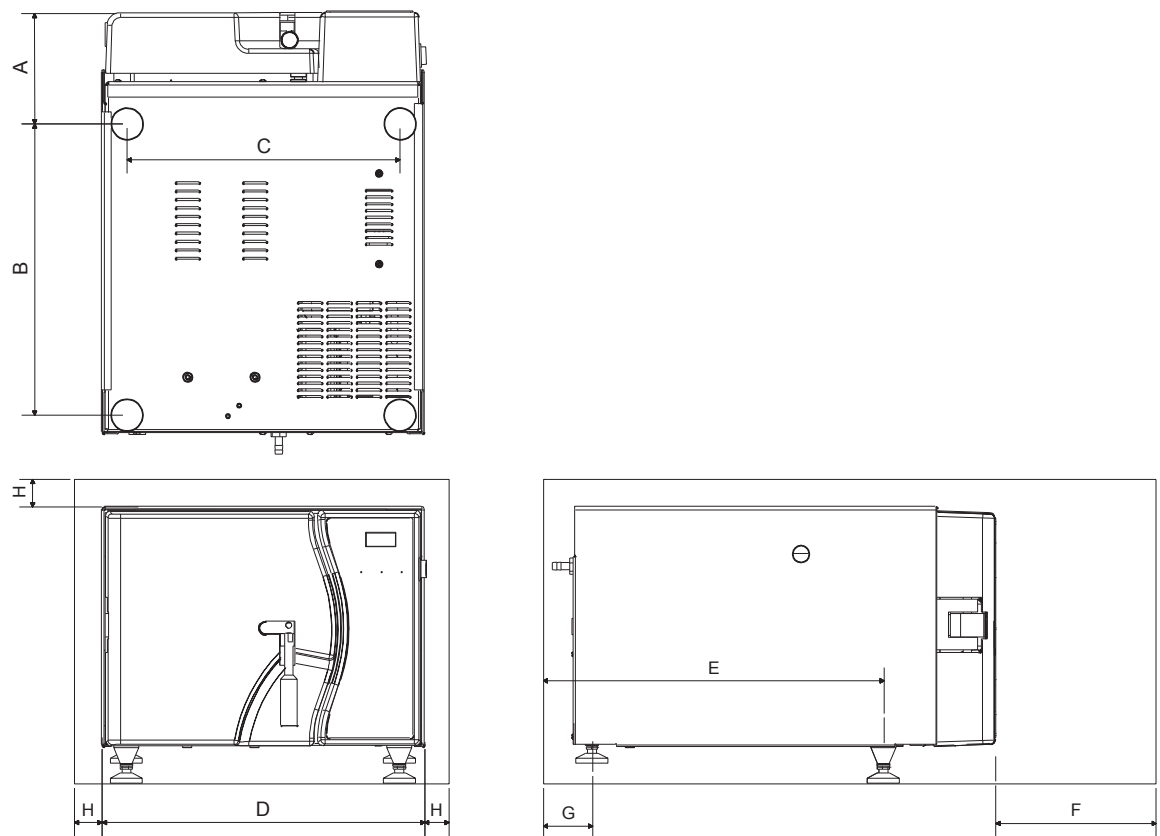A	135 mm
B	347 mm
C	325 mm
D	384 mm
E	420 mm
F Max. hatch opening	300 mm
G	73 mm
H	50 mm

4

OUTPUTS AND INDICATORS LIST

See the folded sheet at the end of the manual.

5

UNPACKAGING

The autoclave is shipped in a suitable package to be transported and moved easily and to protect its contents. The packaging must not be subject to impact, must be handled with care and avoid dropping it or roll it. In case autonomous handling means are not available handle the packaged autoclave always in two persons. Autoclave is placed on wooden pallet and packed with corrugated cardboard internal and external application. For remove the autoclave to its pack please remove first the corrugated cardboard. For lift the autoclave please use the belts.

The autoclave shall have to be handled by at least two people and by using belts only.

Do not lift the autoclave from the inferior part of the door or command panel, this incorrect operation can create problems of a mechanical nature.

ATTENTION: Always conserve original packaging.

Within the package you will find:

- ***USER'S MANUAL:*** to be read with attention and kept in a place available to all operators assigned to sterilization.
- ***CERTIFICATION:*** which must be conserved.
- ***QUICK USE GUIDE:*** should be kept in the vicinity of the machine.
- ***CERTIFICATION OF SAFETY WALVE.***

6**ACCESSORIES**

The autoclave is sold with accessories included in the same package.

N° PIECES	SUPPLIED ACCESSORIES
1	TRAY EXTRACTION AND DOOR ADJUSTMENT WRENCH
3	TRAY (Aluminium anodized – 115 x 17 x 310 mm)
1	DRAIN TANK PIPE
1	SPONGE
1	TRAY HOLDER (Aluminium anodized - 124 x 114 x 310 mm)
1	CONDENSATE DRAIN PIPE
1	WATER FILLING PIPE WITH FUNNEL

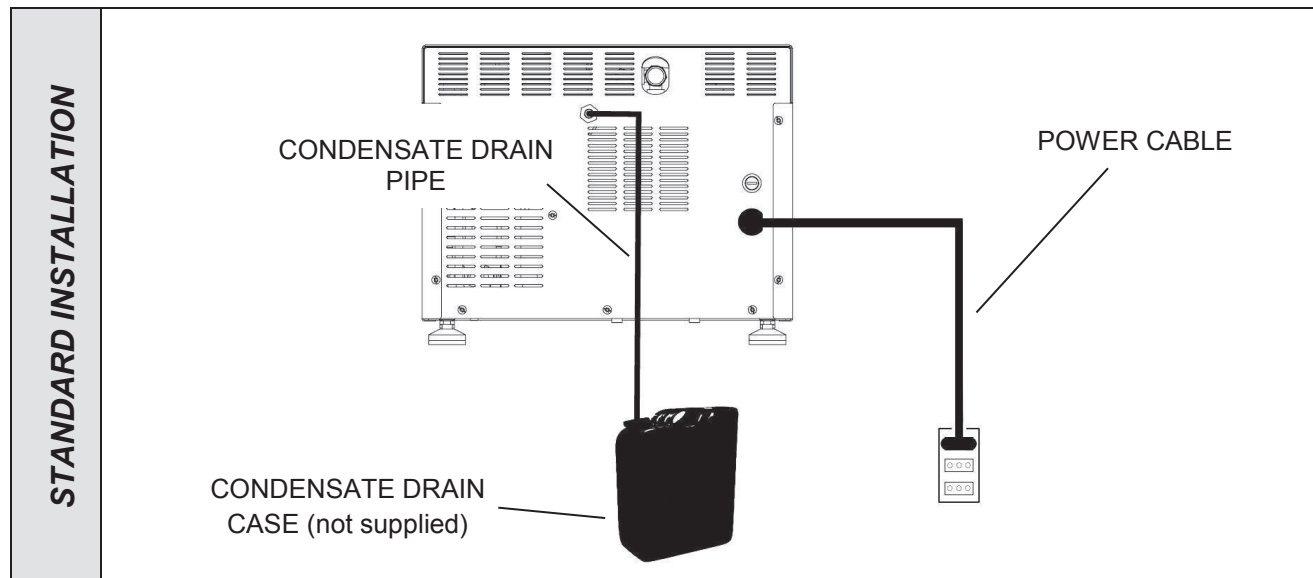
7**INSTALLATION**

The installation is a fundamental operation for a good operation of the appliance. Below are indicate the points to be followed to ensure a good installation.

- 1 – Install the autoclave in environments suitable to sterilization.
- 2 – The premises have to be suitably lit and ventilated, in compliance with regulations in force.
- 3 – Install the autoclave away from heat sources and water splashes.
- 4 – Position the autoclave on a surface, suitable to support the weight (min. 60 Kg.) and with adequate dimensions.
- 5 – Place the autoclave at a height allowing easy intervention by the operator for inspection and clearing of the whole sterilization chamber.
- 6 - Open the door of the autoclave and remove all packages containing the single accessories from inside the sterilization chamber.
- 7 – Leave inside the sterilization chamber only the tray carrier with the trays. All other accessories should be positioned in a separate space available to operators.
- 8 – Don't put anything on the autoclave.
- 9 – Do not lean on the door.

10 – Leave a space of at least 5 cm in the rear of the autoclave, using the spacer and in the sides of the unit to ensure the ventilation required.

11 - Make connections of the condensate drain pipe at the rear of the autoclave.

12 - Always make sure the electrical system to which the autoclave is to be connected is in conformity with the legislation in force and sized to suit the specifications of the said device.

13 - Connect the power supply cord to the socket on the rear panel of the autoclave.

NOTE:

Do not use extensions, reducers or adapters for connection as this could cause micro interruptions with consequent generation of alarm signals.

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Water tank load

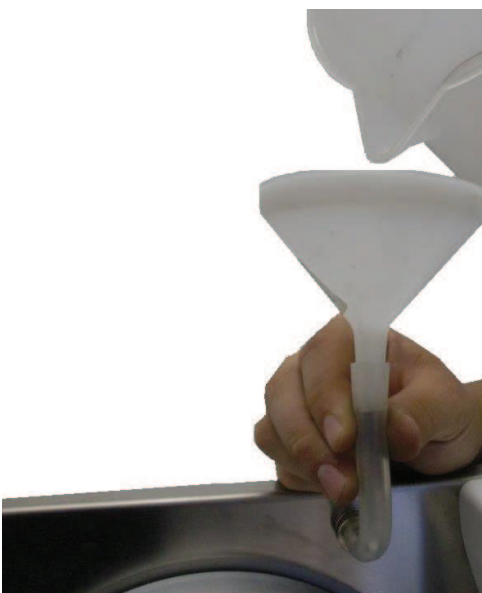
Procedure for the water charge:

- Completely empty the tank
- Fill the carafe up to 1 liter of demineralized water

ATTENTION: Use only demineralized water with conductivity not more than 15 microsimens / cm (see table below "WATER FEATURES TO USE")

- Turn on the tap **08** acting to the ring nut turning in couterclockwise.
- Insert the pipe with funnel and spill the demineralized water (watch water features in the following table).
- Spill all the water in to the funnel.

IMPORTANT: Turn on the tap **08** acting to the ring nut turning in clockwise after the load, in order to avoid steam releases to the frontal part.

WATER FEATURES TO USE

Evaporation residue	≤	10	mg/l
Silicon oxide	≤	1	mg/l
Iron	≤	0.2	mg/l
Cadmium	≤	0.005	mg/l
Lead	≤	0.05	mg/l
Remains of heavy metals apart from iron, cadmium, lead	≤	0.1	mg/l
Chloride (Cl')	≤	2	mg/l
Phosphate (P20s)	≤	0.5	mg/l
Conductivity (at 20°C)	≤	15	μs/cm
Ph value (acidity level)	5 ÷ 7,5		
Appearance	Transparent, clear, without deposits		
Hardness (and alkaline earth ions)	≤	0.02	mmol/l

NOTE: using water with concentrations higher than the one indicated on the present table can reduce considerably the autoclave life causing serious damages to their components and causing the warranty void!

NOTA : Every 5 cycles completely empty the autoclave.

8.2 Carico materiali in autoclave

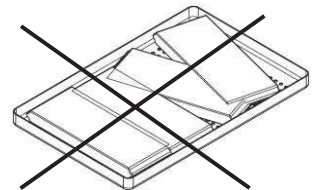
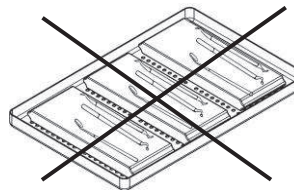
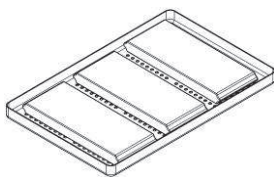
Thanks to the particular characteristics, each cycle of the autoclave ensures the proper drying of the load to be sterilized. It's very important to follow these guidelines for loading the material to achieve optimum results.

Place the holder in a horizontal position respect to the support plane of the machine.

Arranging the materials to be sterilized on the provided trays, as follows:

TOOLS

- do not superpose the materials
- arrange the wrapped materials with the paper side facing upwards
- never bring the materials into contact with the sterilization chamber or the autoclave door
- put scissors and dental forceps with blades open
- when possible, disassemble instruments before placing them in an autoclave.
- avoid the use of lubricants, unless absolutely necessary (in case of water-repellent products, or that prevent the permeation of steam, it reduces the drying process also leading to inadequate sterilization).


8.3 Sterilization cycle

1. Press the button **13** to switch on the autoclave.
2. Open the door turning the lever 04 forward and sliding to the left.
3. Fill the water tank (see details in section 8.1)
4. Press the button **12** to load water in chamber.

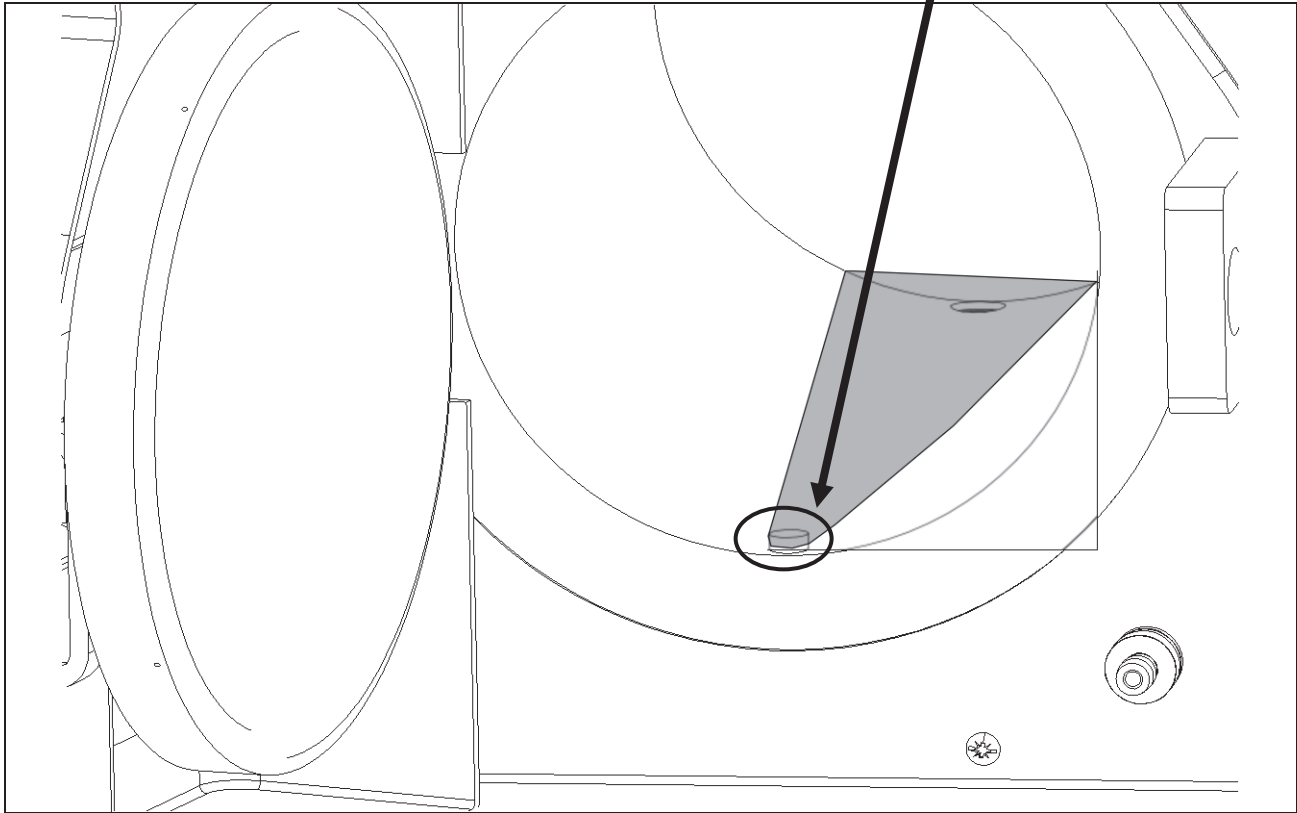WATER LOADED IN CHAMBER MUST ALWAYS GET TO THE METALLIC CATCHER. OTHERWISE DAMAGES TO THE MACHINE OR TO THE INSTRUMENTS CAN OCCUR.

5. Leave the button **12** to stop the water inlet.

ATTENTION: LET ALWAYS WATER INLET CHAMBER BEFORE STARTING THE STERILISATION CYCLE UP!

6. Put the instruments on the trays (see details in section 8.2)
7. Set the required sterilization cycle (*Cycle 121°C or Cycle 134°C*) pressing the button **11**.
8. Close and block the door (sliding the lever to the right and pushing to the front panel) and at the same time let the cycle start. If this is not happening in the stabilized time the autoclave is showing the ERROR 450.

ATTENTION: LET THE CYCLE START UP AND CLOSE THE DOOR AT THE SAME TIME

9. Start the sterilization cycle up pressing the button **10**.

The autoclave is starting the preheating phase (on the display you can read HEATING). When the machine is reaching the necessary pressure for the chosen cycle, the sterilization phase is starting (on the display you can read "STERILIZ"). On the display is shown the remaining sterilization time. When the sterilization cycle is completed, the machine is discharging the pressure and the drying phase is starting (on the display you can read "DRYING"). On the display is shown the remaining drying time. When the sterilization cycle is completed on the display you can read "END".

BEFORE OPENING THE DOOR BE SURE THAT THE CHAMBER IS NOT UNDER PRESSURE.

DO NOT OPEN THE DOOR IF THE PRESSURE VALUE SHOWN ON THE DISPLAY IS HIGHER THAN 0,03 bar

10. Open the door after then you have checked that the chamber is not under pressure.

REMARK 1: the closing bolt is complete with a notch to not allow the door opening if the chamber is under pressure. To make the opening easier push the door toward the chamber;

REMARK 2: if you do not open the sterilization chamber immediately after the end of the sterilization cycle the steam is condensing on the door, so drops will drop while opening the door.

TO AVOID BURNINGS KEEP AT THE RIGHT DISTANCE WHILE OPENING THE DOOR AND KEEP SAFE FROM STEAM LOSSES

11. Wear personal protective equipment in compliance with regulations on safety and hygiene at work. Extract the trays by using the spanner provided, let rest the instruments and store them in environments which are not exposed to contamination.

8.4 Discharge water tank

IMPORTANT

Before filling the tank with demineralized water, drain residual water.

DRAINING WATER TANK

Place a head of the drain tank pipe on the fitting of the tap **07** and the other into a tank located under the level of the shelf on which is positioned the autoclave.

Open the tap **07** rotating anticlockwise the ring nut.

Wait the complete emptying of the tank, (rotating clockwise) completed the step of draining water.

Proceed with the load of demineralized water.

	STERILISATION TIME [minutes]	DRYING TIME [minutes]	WORKING PRESSURE [kPa Relative]
121°C	22,5	20	130 ± 20
134°C	8	15	225 ± 20

The choice of the sterilisation cycle must be based on the data supplied by the producer of the object you want to sterilise.

TECNO-GAZ is at your service to provide sterilization tests for the management of your sterilization routine

9.1 Cycles backups on USB key

The autoclave records cycle data on an internal memory containing 10 records.

It is possible to save the recorded cycles on a USB memory, using USB port on the side of main board (Pos. 14).

The USB memory must be formatted in FAT form.

In order to save the data follow these steps:

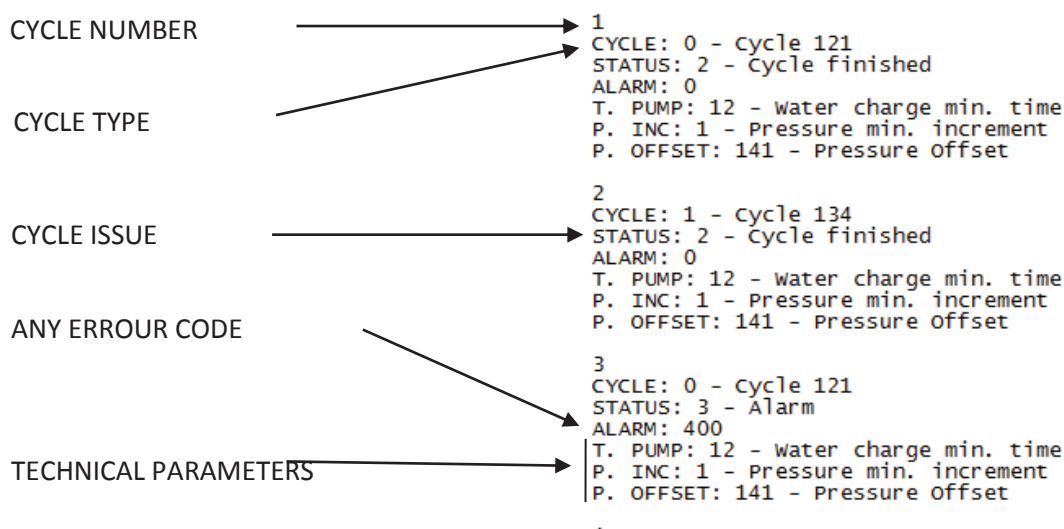
- If the autoclave is turned on, turn off it (by a general switch Pos. 13)
- Connect the USB memory in the outlet (Pos. 14)
- Push and hold the button SELECT (Pos. 11)
- Switch on the autoclave (by a general switch Pos. 13) and hold the button SELECT (Pos. 11)

The display shows the following screens:

LOG USB, KEY DETECTED, OPENING FILE, MOUNT OK, READING MEMORY, FILE SAVED!

Once visualized "FILE SAVED!" is possible switch off the device and pull the USB memory.

On the USB memory is saved a text file with the follow formatting:

10

MAINTENANCE

Correct maintenance of the autoclave assures correct functioning of it and a secure saving in terms of time and costs for assistance and maintenance. The following operations are compulsory feasible by operators.

Cleaning of chamber

Every 20 cycles or one a week

Clean periodically the chamber, remove eventual deposits or debris, thus avoiding the introduction in the discharging circuit of material which can cause obstructions. For correct cleaning use only demineralized water and the abrasive sponge provided.

To execute absolutely with cold chamber to avoid burns – Never use solvents, detergents, chemical solutions, descaling agents or other similar products.

Cleaning of chamber filter

Every 20 cycles or once a week

Pull up the filter, paying attention not to damage it, wash with demineralized water and dry with cloth. Then replace in the seat, making sure that protrudes about 15 mm.

Cleaning of tray and tray holder

Every 20 cycles or once a week

Clean with sponge supplied (non abrasive side) and demineralized water.

Cleaning of the door gasket

Every 20 cycles or once a week

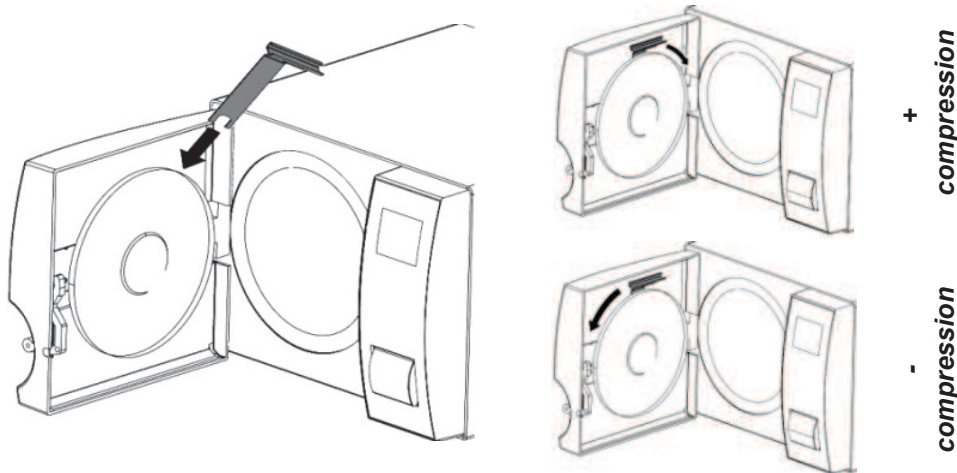
Periodically remove eventual residuals which are deposited on the circumference of the seal using water and the sponge provided (non abrasive side), or else a moistened cloth.

Door regulation

Every 2 months

To regulate the door lock pressure, it is necessary to adjust the regulator of the same door using the double function wrench (extraction tray + wrench) provided.

Rotate clockwise to increase the pressure of closing. Rotate anticlockwise to decrease the pressure of closing.

The following steps are recommended by the manufacturer. These operations must be performed only by trained technicians authorized by the manufacturer.

Replacement of the water loading filter	<i>every 1 year / every 1200 cycles*</i>
Cleaning Y brass filter downstream of the radiator	<i>every 1 year / every 1200 cycles*</i>
EV4 cleaning	<i>every 1 year / every 1200 cycles*</i>
Fan cleaning	<i>every 1 year / every 1200 cycles*</i>
Control integrity of the pneumatic circuit	<i>every 1 year / every 1200 cycles*</i>
Control integrity of the electric circuit	<i>every 1 year / every 1200 cycles*</i>
Control thermal insulation of the chamber	<i>every 1 year / every 1200 cycles*</i>
Control tightening screws	<i>every 1 year / every 1200 cycles*</i>
Safety valve check	<i>Follow the guidelines of the manufacturer (see instructions)</i>
State of tank check	<i>every 1 year / every 1200 cycles*</i>

* valid the condition of the reached first

Regional/national regulations fix the mandatory of autoclave validation that checks sterilization parameters. Periodic validation must always be performed by trained technicians using appropriate measurement instruments. In the case of non-mandatory, validation is a voluntary operation which aims to certify that the autoclave is fully functional and fit for sterilization of loads for which it was designed and built.

If an alarm message is displayed the cycle is to be considered aborted: it will be necessary to repeat all the preparation and sterilization operations.

If an error message appeared on the display during the sterilization phase, the cycle failed and the instruments in the chamber were not sterilized.

To RESET the error message, push the button 11

MESSAGE	CAUSE
ERROR100	The sterilization cycle was interrupted by the operator pressing the button 10 .
ERROR120	Low pressure in the chamber while sterilizing
ERROR125	High pressure in the chamber while sterilizing
ERROR140	The pressure did not increase properly during the preheating phase or the machine could not discharge the pressure at the end of the sterilizing cycle. If this is happening even if the water is in the chamber, dry and empty the sterilization chamber, reset the alarm, let the water in and restart the cycle up
ERROR199	The power was disconnected while the sterilization cycle
ERROR400	The cycle was started up without let water in the chamber. Dry and empty the sterilization chamber, reset the alarm, let the water in and restart the cycle up.
ERROR401	The cycle was started up but 5 min went from the water inlet in the chamber. Dry and empty the sterilization chamber, reset the alarm, let the water in and restart the cycle up
ERROR450	The cycle was started up but the machine was hot. After the door closing the machine let the water out of the chamber. Reset the alarm, wait until when the machine is cold, Dry and empty the sterilization chamber, let the water in and restart the cycle up
ERROR700	During the preheating phase, in 40 minutes was not reached pressure value established for sterilization. If this is happening even if the water is in the chamber, dry and empty the sterilization chamber, reset the alarm, let the water in and restart the cycle up
ERROR760	Failure in the device for the pressure level
ERROR800	The pressure did not go down properly during the drying phase.

If the problem is coming out even if you carried out the listed operation, please contact the assistance

12**SOLUTION TO OPERATING PROBLEMS**

In most cases alarms or errors are caused by lack of attention or lack of familiarity with some technical and operational aspects. You will find below the list of some anomalies with associated solutions.

12.1 The autoclave does not dry materials correctly

- Non-original trays, without holes or with different holes made of different material have been used. *It is advisable to use original trays only.*
- Instruments have not been arranged correctly. (instructions of par.8.2)

12.2 The autoclave chamber has turned white

- Change immediately the type of water used, use exclusively demineralized or distilled water, as indicated in the previous chapters and then clean the chamber.
- The whitish colour may be a consequence of the evaporation of organic materials from the instruments. Instruments should undergo a more suitable and deeper cleaning.

12.3 The autoclave chamber has bluish-green stains

- Instruments have not been cleaned correctly after the cleaning, rinse more carefully, if the stains remain apply to the phone technical service.

12.4 The sterilization cycle interrupts without any patent reason

- Check whether the autoclave is connected to the power mains through extensions, reducers or adapters, if so remove such accessories and connect the autoclave directly to the power socket.

12.5 Stains on instruments

- Instruments get yellow due to the deposit of residual chemical fluid which has fixed on the instruments due to heating. The rinsing was not suitable.
- The sterilization chamber has yellow stains. Some instruments with traces of chemical fluid have been loaded into the chamber, such fluid has fixed due to heating. The rinsing was not adequate.
- Instruments have whitish stains; the rinsing has been done with very limy water and the instruments have not been carefully dried. For the final rinsing, we suggest the use of demineralized water and a careful drying of the instruments
- Instruments have blackened since the material they are made of contain a high percentage of carbon.

13

PROCEDURE FOR SERVICE AND ASSISTANCE

In case of failure, review, validation, contact the service centers
TECNO-GAZ S.p.A.

**See annex for Authorized Service Centers
Cod. 0Z00H0002**

Assistance will assess the return at headquarters or with the intervention of a technician, and having viewed the machine in order to draw up a cost estimate, which will be forwarded to the distributor customer who will forward it to the final customer, for acknowledgement and signing.

After receiving prior written acceptance of the cost estimate, the autoclave will be serviced and reshipped according to times indicated on the cost estimate.

In case the autoclave must be shipped for repairs, controls, reactivation, revisions, validations follow the obligatory indications below:

- Use the original packaging; if this is no longer in your possession, use adequate packaging. The merchandise travels at risk to the sender.
- Ship the autoclave only (do not include any component contained in the accessories kit).
- Carefully clean the sterilization chamber and autoclave in general before shipping. In case it arrives dirty or with residual the autoclave will be returned without being repaired, or it will be put through a cleansing action and disinfection.
- Always empty the water tank through the attachment **07** located on the back of the autoclave.
- Indicate by letter and insert in the package a document which indicates precisely the irregularity or service desired.
- Ship at your expense, otherwise you will be billed for shipping

All non-original packaging which arrives will be disposed

Autoclave will be returned with new and original packing to insure maximum protection for your autoclave during shipping. Cost of packaging will be charged to client.

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***Voir svp la liste des S.A.V. en annexe